



ClearView
Veterinary Imaging



CorridorVet

USER GUIDE

• WWW.CORRIDOR.VET •



support@corridor.vet
(800) 208-2403



How to submit diagnostic imaging for telespeciality review

South Sound Veterinary Imaging Radiology and Cardiology w/ CorridorVet

1. Go to pacs.corridor.vet → Login using your credentials

- **Direct DICOM send (preferred)** Studies can be submitted from direct DICOM send from your equipment and/or PACS system (preferred) or DICOM images can be imported using the import study function on the bottom of the toolbar
- **Import DICOM**- manually import dicom images using the import function on the right toolbar



- **For non-DICOM files ONLY** (ie. no associated DICOM images, such as an ECG or clinical review cases)- manually import non-dicom images using the non-DICOM study on the right toolbar



2. Select “studies” or “needs attention” from menu bar on the left

3. Locate study to be submitted on table- wait for loading

4. Select the **Consult** action button  to the left of the patient name on the action tool bar



5. Fill out consult form

1. Select desired reporting company from dropdown

- Default will be South Sound Veterinary Imaging, for cardiology, select Cardiology-SSVI from dropdown.

2. (Optional) Specific radiologist request (First available is recommended, specific requests could result in significant delays in turnaround times).

3. Priority- STAT (1hr), Priority(12hr) or Standard (48hr)- for SSVI radiology only, Cardiology is standard only as of current.

4. (Optional) Billing code, this will be confirmed by reporting radiologist. This supports multiple entries


- EMPLOYEE PET MUST BE SELECTED if submitted case is owned by employee for discount to be applied

5. Fill out patient information and study description in form (*** fields are mandatory- send will not be active until completed**)

6. Send form (Or Save)

- *If send is grayed out*, make sure all fields with * are complete.
- Save form will NOT send the form but will save form details for later submission.

- To send after save – click  on main actions buttons or reopen consult  and hit send at Bottom of the sheet.

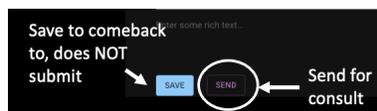

If you accidentally submit a case for review and need to edit

- Go to “pending” or locate study on the “studies tab and select the “clear” action button . This will return the study to the needs attention tab and you can edit details as needed.
 - (only applicable if case has not yet been claimed by radiologist, if claimed by radiologist already, please call/text us to sort out.

How to Access Completed Reports

South Sound Veterinary Imaging – CorridorVet

All reports will be emailed to login email account following completion from noreply@corridor.vet

1. Make sure you are logged in at pacs.corridor.vet
2. Select “Studies” or “Completed”
3. Select PDF button from action tool bar 
4. PDF preview will populate – select download in bottom right corner of PDF preview  to download pdf for import into record, email, etc

Guide to action buttons

Available actions vary depending on case status

“Needs attention”- Cases not yet submitted for consult

-  Zip file- download
-  Viewer
-  Rename study
-  Telespecialty Consult Request Submission
-  Submit not submitted consult (if not sent from request form)

“Pending”- Cases submitted for teleconsultation but not yet completed

-  Zip file- download
-  Viewer
-  Rename study
-  Undo consult- Works if not yet claimed by radiologist
Will go back to needs attention queue for editing (previous information saved)

“Complete”- Cases submitted for teleconsultation and completed

-  Zip file- download
-  Viewer
-  Rename study
-  **Completed Report**